



Clinical Results

Before & After Photos
With Long Term Follow-Up



EndyMed™
THE 3DEEP® COMPANY

EndyMedPRO™

The sleekly designed EndyMed PRO system is multi-application, upgradeable and available with multiple handpieces, customized to different applications and treatment areas. EndyMed PRO utilizes proprietary 3DEEP® RF technology and has an easy-to-use touch screen interface.



3DEEP® Body Contouring Handpiece



3DEEP® Body Tightening Handpiece



3DEEP® Facial Tightening Handpiece



3DEEP® Fractional Skin Resurfacing Handpiece




3DEEP® Technology

EndyMed's proprietary 3DEEP® technology is a multi-source, phase-controlled radiofrequency energy source that is state-of-the-art, safe and effective.

3DEEP uses an array of multiple RF sources and a sophisticated algorithm to manage the phase of the energy flowing between sets of electrodes. The repelling forces between adjacent electromagnetic fields drive the energy vertically into the target tissue, reducing significantly the energy flowing along the skin's surface (and any potential side effects) and eliminating the need for cooling.

Energy penetration depth of 3DEEP RF

3DEEP RF – focused and contained deep energy flow with no need for cooling

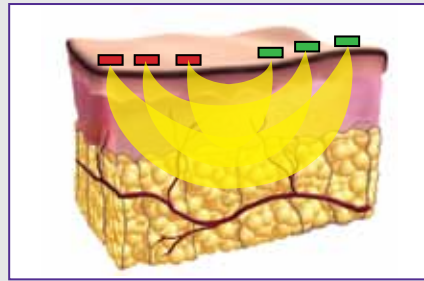


3DEEP® Fractional Skin Resurfacing

3DEEP Fractional Skin Resurfacing provides a smoother, brighter skin surface (stratum corneum and epidermis) and reduced hyperpigmentation (epidermis). Simultaneously, volumetric deep dermal heating reaches the collagen fibers causing an immediate and long term tightening effect with a perceived reduction in the depth of atrophic acne scars. 3DEEP Fractional Skin Resurfacing effectively treats all skin layers for an optimal result.

Fractional application of 3DEEP RF energy

Fractional effect
on the epidermis

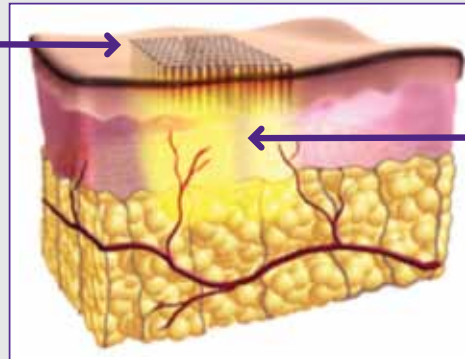

Full volume of deep
dermal heating

Simultaneous fractional micro-ablation
of the epidermal layer and deep
dermal volumetric heating.



↔ 3DEEP Facial Tightening



Before treatment



6 months follow up



Photographs courtesy of
I. Vider, M.D., Medical OR, Hertzlia, Israel

3DEEP Facial Tightening



Before treatment



Immediately after one treatment

Photographs courtesy of
Flor Mayoral, M.D., Miami, Florida

↔ 3DEEP Facial and Neck Tightening



Before treatment



After 6 treatments

Photographs courtesy of
Prof. Daniel Cassuto, Milan, Italy



3DEEP Facial and Neck Tightening



Before treatment



After 6 treatments

Photographs courtesy of
Prof. Daniel Cassuto, Milan, Italy

↪ 3DEEP Facial and Neck Tightening



Before treatment



3 months follow up



5 months follow up



Photographs courtesy of
I. Vider, M.D., Medical OR, Hertzlia, Israel

↔ 3DEEP Facial and Neck Tightening




Before treatment



6 months follow up

Photographs courtesy of
I. Vider, M.D., Medical OR, Hertzlia, Israel

↔ 3DEEP Facial and Neck Tightening

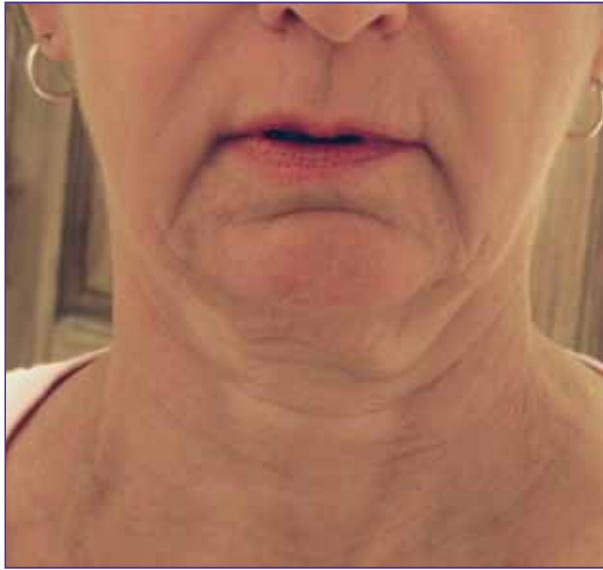

Before treatment



After treatment

Photographs courtesy of
Fiona Wright, M.D., Plano, Texas, USA



3DEEP Facial and Neck Tightening



Before treatment



After treatment

Photographs courtesy of
Fiona Wright, M.D., Plano, Texas, USA

↔ 3DEEP Facial and Neck Tightening



Before treatment



6 months follow up

Photographs courtesy of
I. Vider, M.D., Medical OR, Hertzlia, Israel

